Instructions of Use

Gebrauchsanweisung Istruzioni per l'uso Gebruiksaanwijzing Instruções de utilização Käyttöohje Návod k použití Mode d’emploi

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Instructions of Use

	Manufacturer
	„DO NOT REUSE“
	Batch code
	DATE OF MANUFACTURE
	End of shelf life yyyy/mm
	See information supplied by the manufacturer
	Temperature range of storage conditions
	Maximum relative humidity of storage conditions

Contact info:
Europe: Alpha Solution GmbH Goldsteinstraße 87a D-50968 Cologne
Asia: Shandong Haichun Bio-Technology Co.,LTD No. 1789, Fuzhou Rd, Industrial Development Zone, Heze, Shangdong

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<p>Indications for use The mask is intended to be used for protection against solid and oily particulates as well as liquid and microorganisms such as coal dust and cement dust. It is also single-use.</p> <p>Limitations</p> <ul style="list-style-type: none">It is not suitable for working places where open flames exist (such as welding,casting, etc.). It is further not suitable for protective use in an oxygen deficient or toxic gas environment. <p>Checks prior to use Please check before use whether the packaging is damaged, the mask body is damaged, deformed or has other obvious defects.</p> <p>Fitting</p> <ol style="list-style-type: none">Keep the nose bridge upward and use your hands to extend the straps. Place the respirator on your face to cover the nose, mouth and chin. Place each strap around one ear. Adjust the nose bridge so the respirator lies firmly on your face.
<p>Cautions when in use:</p> <ol style="list-style-type: none"> Please check whether the mask fits prior to use. The mask is only for single use, do not use repeatedly. Please do not use the mask under conditions of low air quality, such as seriously contaminated air or oxygen deficiency. Please note that scars, facial hair and other unevenness along the sealing surface can endanger safe use. Do not use the mask in an explosive atmosphere. After use, please discard the mask following local regulations. <p>Storage Condition: The product should be stored in a well-ventilated, dark and dry environment. Keep away from fire, pollutants and possible pollution sources. Also, the storage temperature should be between -20°~60°, and the relative humidity : <80% RH</p> <p>Shelf-Life: The Protective Mask is valid for 2 years under the above storage conditions.</p>

Kontakt:
Europa: Alpha Solution GmbH Goldsteinstraße 87a D-50968 Cologne
Asien: Shandong Haichun Bio-Technology Co.,LTD No. 1789, Fuzhou Rd, Industrial Development Zone, Heze, Shangdong

Nome do produto: Semi-máscarasiltrantes
Class.: FFP2 NR Modelo: HERA
FFP2 NR de inido na EN 149:2001+A1:2009

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